

ARTICLE	FIT LADY PATCH
MANUFACTURER	D.FENSTEC Srl
TYPE OF PRODUCT	Analgesic patch with FIT Therapy technology : it is able to reduce the pain symptoms with recovery of the injured function.
CLASSIFICATION	CE Medical Device Class I - according to Reg. UE 2017/745.
AVAILABLE SIZES AND FORMATS	Big patch 12,5 x 5,6 cm - Small patches 5,6 x 3,7 cm <ul style="list-style-type: none"> · Envelope 2 kits (art. FIT LADY 2K) <ul style="list-style-type: none"> · with hang-hole - 9,9 x 20 cm · without hang-hole - 9,9 x 17,5 cm
PRODUCT DESCRIPTION	<p>The system consists essentially of two elements: a non-woven polypropylene fabric and an acrylic adhesive mass.</p> <p>The inks contain an appropriately treated titanium dioxide powder, which is able to reflect the far infrared waves that the human body naturally emits, leading to an improvement in blood microcirculation.</p> <p>The Titanium Dioxide used is not classified as a nanomaterial.</p> <p>The product is not a thermal patch, therefore it does not self-heat to achieve the intended effect. However, due to the effect on the local microcirculation, in some cases a slight heat may be detectable in the area of application.</p>
MATERIALS AND PACKAGING	<p>The device consists of the following materials:</p> <ul style="list-style-type: none"> · Non-woven 100% Polypropylene · Acrylic adhesive mass · Ink colours · Titanium dioxide <p>Primary packaging: BOPP film blister. Secondary packaging: custom FIT Therapy cardboard.</p>
PATENT AND BRAND	FIT THERAPY brand product protected by trade secret.
USE	<p>Remove the patch from the base and apply it to the indicated area, ensuring that skin is clean, dry and hair-free.</p> <p>Wait for approximately two minutes before moving to ensure the patch adheres fully to the skin</p>



DISCLOSURE MATERIAL RESERVED FOR SPECIALIZED PROFESSIONAL CATEGORIES




Is a trademark of **D.FENSTEC Srl**
Viale Verona, 74 – 36077 Altavilla Vicentina (VI)
Tel. +39 0444 17 50 103 – info@fit-italy.com
www.fittherapy.biz



Company with **ISO 9001** and **ISO 13485 for MD** certified Quality Management System



FIT LADY PATCH

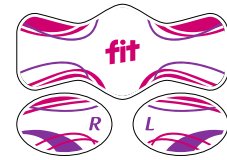

WARNINGS

The products are non-sterile and **do not contain any form of medication.**

Patches are not to be used on wounds or on broken or reddened skin as they may cause infection.

If you suffer from specific blood circulation problems (especially related to microcirculation) or muscular problems, do not use the product without first consulting your GP as it could be ineffective and/or lead to a worsening of the pathological condition.

Do not apply a patch if it shows any sign of wear or defects.

Hypersensitivity phenomena may occur at the site of application of the patch, which may manifest with the appearance of itching, redness, or real erythema sometimes with the appearance of blisters.

If itching/burning occurs after the application of the patch, it is recommended to remove the patch immediately.

If the user is a child or a pregnant woman, it is recommended to consult a doctor before using the product, to avoid possible unwanted side effects.

Do not use the same patch for more than 5 days. In case of continuous applications (even of the same package), replace the patch with a new one as it may lead to a reddened skin condition.

Do not use the product after the expiry date, as it may lose its primary characteristics.

The effectiveness of the patch is not guaranteed in case it is reused.

In case of a serious accident, the user shall report the event directly to the manufacturer and to the competent authority of the Member State where the user is established.

DURATION OF USE

Keep the patch on for 5 days. The patches will continue to work even if they get wet (e.g. in the shower).

EXPIRY

5 years from manufacturing date.

STORAGE

Store between +5°C and +30°C. The packaging will in any case provide the necessary protection to prevent the product from being damaged during movement.



DISCLOSURE MATERIAL RESERVED FOR SPECIALIZED PROFESSIONAL CATEGORIES



Is a trademark of **D.FENSTEC Srl**
 Viale Verona, 74 – 36077 Altavilla Vicentina (VI)
 Tel. +39 0444 17 50 103 – info@fit-italy.com
 www.fittherapy.biz



Company with **ISO 9001** and
ISO 13485 for MD certified
 Quality Management System